



Chuangxingwell

# Free Triiodothyronine (FT3) Rapid Test Kit Instructions for Use



A rapid test for quantitative detection of FT3 in human serum or plasma with the use of Immunofluorescence Analyzer.

- ▶ For *in-vitro* diagnostic use



## PACKING SPECIFICATION

25 Tests/ Kit

## INTENDED USE

The FT3 Rapid Test Kit is based on Fluorescence Immunoassay for the quantitative detection of free Triiodothyronine (FT3) in human serum or plasma. The test is intended for healthcare professionals use.

## SUMMARY

The physiological actions of thyroid hormones can be categorized as growth and development and control of metabolic processes in the body. Hypothalamic-pituitary-thyroid axis can control the synthesis, release and function of thyroid hormone. It's secreted from the hypothalamus Thyrotropin releasing hormone (TRH) stimulates the synthesis and release of thyrotropin or TSH. In turn, TSH Stimulate the synthesis, storage, secretion and metabolism of thyroxine (T4) and triiodothyronine (T3). In the blood, there are free and combined forms of T4 and T3. In blood circulation, more than 99% of T4 and T3 bind to carrier proteins. The remaining Less than 1% of T4 and T3 are free. Such unbound or free hormone levels are associated with thyroid function in most humans. It depends on the energy state.<sup>1,2</sup> Free T3 and free T4 regulate normal growth and development by maintaining body temperature and stimulating heat generation. In addition, FT4 with FT3 also affects all aspects of carbohydrate metabolism and some aspects of fat and vitamin metabolism. Fetal and Thyroid hormones are also needed for newborn development.<sup>1,2</sup>


## PRINCIPLE

The FT3 Rapid Test Kit detects FT3 based on Fluorescence Immunoassay. The specimen moves through the strip from sample pad to absorbent pad. FT3 in the specimen will compete with the T3 antigen coated on the strip. The less FT3 in the specimen, the more fluorescent microspheres conjugated with anti-T3 antibodies can be captured by the T3 antigen coated on the strip. The concentration of FT3 in the sample is inversely related to the intensity of the fluorescent signal captured on the T line. According to the fluorescence intensity of the test and the standard curve, the concentration of FT3 in the sample can be calculated by analyzer to show FT3 concentration in the specimen.

## REAGENTS

The test KIT contains T3 antigen and T3 antibody.

## MATERIALS PROVIDED

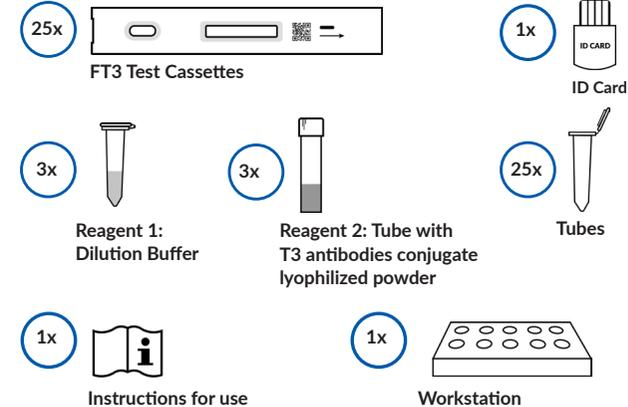

## MATERIALS REQUIRED BUT NOT PROVIDED

- ▶ Immunofluorescence Analyzer
- ▶ Pipettes
- ▶ Centrifuge
- ▶ Specimen Collection Containers
- ▶ Timer

## PRECAUTIONS

1. For professional *in-vitro* diagnostic use only.
2. Do not use after the expiration date indicated on the package. Do not use the test if the foil pouch is damaged. Do not reuse.
3. Avoid cross-contamination of specimens by using a new specimen collection container for each specimen obtained.
4. Do not eat, drink or smoke in the area where the specimens and tests are handled. Handle all specimens as if they contain infectious agents. Observe established precautions against microbiological hazards throughout the procedure and follow standard procedures for proper disposal of specimens. Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
5. Do not interchange or mix reagents from different lots.
6. Humidity and temperature can adversely affect results.
7. Used testing materials should be discarded in accordance with local regulations.
8. Read the entire procedure carefully prior to any testing.
9. The FT3 Test Cassette should only be used with the analyzer by approved medical professionals.

## STORAGE AND STABILITY

1. The test should be stored at 4 - 30 °C until the expiration date printed on the sealed pouch. Shelf life is 24 months.
2. The test cassette must remain in the sealed pouch until use and must be used within 1 hour if opened.
3. Do not freeze.
4. Care should be taken to protect the components of the test from contamination.
5. Do not use if there is evidence of microbial contamination or precipitation. Biological contamination of dispensing equipment, containers or reagents can lead to false results.

## SAMPLE COLLECTION AND PREPARATION

1. Collect the specimen according to standard procedures.
2. Do not leave specimens at room temperature for prolonged periods. Serum and plasma specimens may be stored at 2 - 8 °C for up to 7 days, for long term storage, specimens should be kept below -20 °C.
3. Bring specimens to room temperature prior to testing. Frozen specimens must be completely thawed and mixed well prior to testing. Avoid repeated freezing and thawing of specimens.
4. EDTA K2, Heparin sodium, Citrate sodium and Potassium Oxalate can be used as the anticoagulant for collecting the specimen.
5. Before performing the test, please balance the sample to room temperature (15 - 30 °C). Frozen specimens must be completely thawed and mixed well prior to testing.


## TEST PROCEDURE

Refer to Immunofluorescence Analyzer User Manual for the complete instructions on use of the Test. The test should be in room temperature.


Allow the test cassette, specimen and buffer to reach room temperature (15 - 30 °C) prior to testing.




Turn on the Analyzer power.

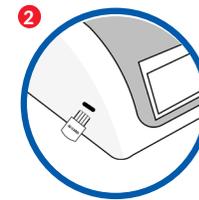


Take out the ID card and insert it into the Analyzer ID Card Slot. Choose test mode and/or sample type according to needs.

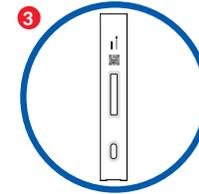


Remove the test cassette from the sealed foil pouch and use it within 1 hour. Best results will be obtained if the assay is performed immediately after opening the foil pouch. Place the test on a flat and clean surface.

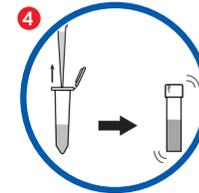

Pipette 900 µL of reagent 1 into the reagent 2 tube and dissolve the lyophilized powder thoroughly. **Note: After dissolution, the dissolved reagent 2 could be stored at 2-8 °C for 30 days or stored at -20 °C for long period.**

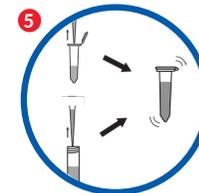

Pipette 75 µL of the dissolved reagent 2 and 75 µL of serum or plasma into empty tube to mix the specimen and the dissolved reagent 2 well. Leave the mixture for reaction of 5 minutes.

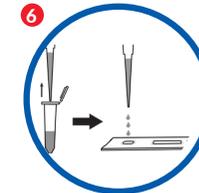

Then pipette 75 µL diluted sample into the sample well of the cassette. Start the timer at the same time.



15 min

Test results should be interpreted at 15 minutes with the use of Immunofluorescence Analyzer.

**Caution:** There are different test modes of the Immunofluorescence Analyzer. The difference between them is incubation of the test cassette is outside or inside the analyzer. Choose test mode accordingly and confirm sample type. Consult the user manual of the analyzer for detailed operation information. Operator must consult the Immunofluorescence Analyzer User Manual prior to use and become familiar with the processes and quality control procedures.

## INTERPRETATION OF TEST RESULTS

### Results read by Immunofluorescence Analyzer.

The result of test for FT3 is calculated by Immunofluorescence Analyzer and display the result on the screen. For additional information, please refer to the user manual of Immunofluorescence Analyzer.

Linearity range of FT3 is 1.5 - 46 pmol/L (0.97 - 29.8 pg/mL).

Normal Reference range (adult): 2.6 - 5.7 pmol/L (1.7 - 3.7 pg/mL).

Conversion factor as unit of pmol/L (SI unit) = 1.54\*pg/mL.

### QUALITY CONTROL

Each FT3 Rapid Test Kit contains internal control that satisfies routine quality control requirements. This internal control is performed each time a sample is tested. This control indicates that the test cassette was inserted and read properly by Immunofluorescence Analyzer. An invalid result from the internal control causes an "N/A" message on Immunofluorescence Analyzer. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control failure. Review the procedure and repeat the test with a new test. If the problem persists, discontinue using the test kit immediately and contact your local distributor.

### LIMITATIONS

1. The FT3 Rapid Test Kit is for *in-vitro* diagnostic use only.
2. The test should be used for the detection of Triiodothyronine (Free T3) in serum or plasma specimens only.
3. The FT3 Rapid Test Kit will only indicate the presence of FT3 in the specimen and should not be used as the sole criteria of diagnosis of hyperthyroidism or hypothyroidism.
4. Like with all diagnostic tests, a confirmed diagnosis should only be made by a physician after all clinical and laboratory findings have been evaluated.
5. For patients receiving high-dose biotin (ie >5 mg/day), samples can be collected at least 8 hours after the last biotin dose.

### EXPECTED VALUES

Concentrations	Clinical Reference
<2.6 pmol/L (1.71 pg/mL)	Low
2.6 - 5.7 pmol/L (1.71 - 3.7 pg/mL)	Normal
>5.7 pmol/L (3.7 pg/mL)	High

Each laboratory should determine the applicability of the reference range through experiments, and establish its own reference value range if necessary to ensure that it can correctly reflect the situation of a particular population.

### PERFORMANCE CHARACTERISTICS

#### 1. Method comparison

The assay was compared with commercial CLIA test kit with 140 samples. The correlation coefficient ( $R^2$ ) is 0.9925

#### 2. Accuracy

The test deviation is  $\leq \pm 15\%$ .

#### 3. Linearity range

1.5 - 46 pmol/L (0.88 - 29.8 pg/mL),  $R \geq 0.990$

#### 4. Precision

##### Intra-lot precision

Within-run precision has been determined by using 10 replicates of 2 different concentration of FT3. C.V. is  $\leq 15\%$ .

##### Inter-lot precision

Between-run precision has been determined by using 10 replicates for each of three lots using 2 different concentration of FT3. C.V. is  $\leq 15\%$ .

#### 5. Interfering substances

The following substances do not interfere with the test results at the indicated concentrations: Ascorbic Acid at 200 mg/L, Hemoglobin at 10 g/L, Triglyceride at 30 g/L, Bilirubin at 1,000 mg/dL, Uric Acid at 200 mg/L.

## BIBLIOGRAPHY

1. Gornall, AG, Luxton, AW, Bhavnani, BR. Endocrine Disorders. In Applied biochemistry of clinical disorders. 1986; 305-318. Philadelphia, PA: J. B. Lippincott Co.
2. White, GH. Recent advances in routine thyroid function testing. CRC - Critical Reviews in Clinical Laboratory Sciences. 1987; 24: 315-362.

## SYMBOLS

	Caution		In Vitro Diagnostic Medical Device
	Manufacturer		Date of Manufacture
	CE Marking		Do Not Re-use
	Keep Dry		Keep Away From Sunlight
	Batch Code		Do Not Use if Package is Damaged
	Catalogue Number		Contains Sufficient for <n> Tests
	Use-By Date		Temperature Limit
	Consult Instructions for Use		Authorized representative in the European Community



#### Manufacturer

Jinan Chuangxingwell Biotech Co., Ltd.  
Add.: Building 1-2B-402, Liando U Valley,  
Gangyuan Sixth Road, Jinan Zhangjin  
Comprehensive Bonded Zone, Gaoxin  
District, Jinan city, Shandong Province,  
China  
Tel: +86-0531-88894020  
Email: s@1stiot.com



#### Riomavix S.L.

Add.: Calle de Almansa 55, 1D, Madrid 28039 Spain  
E-mail: leis@riomavix.com  
Tel.: +34 658 396 230  
(SRN: ES-AR-000001202)